



**The Power to Eliminate
The Promise to Protect**



Disinfection & Sterilization

ZERO COMPROMISE

MIREY 70% IPA

Isopropyl Alcohol IP – 70% Solution

Effective Surface Disinfection &
Equipment Cleaning

Composition

- Isopropyl Alcohol IP – 70%

Key Properties

- Ready-to-use disinfectant
- Rapid Evaporation

Usage Instructions

For Surface:

- Wipe the surface to physically remove any contaminants
- Spray from a distance of 6–8 inches on surface
- Apply directly onto surfaces ensuring even coverage
- Keep surfaces visibly wet for effective disinfection
- Always close the nozzle after use

For Electronic Equipment

- Spray MIREY 70% IPA onto a **sterile gauze** to moisten it
- **Do not spray directly on screens or sensitive electronic equipment**
- Wipe the screen or surface gently **in one direction only** to avoid streaks and static damage

Precaution

- Avoid use on alcohol-sensitive materials such as **acrylic glass**

Pack Size



Bottle	Carton
250ml	20 Bottles

INTRODUCING

UPCOMING PRODUCT RANGE

Sr. No	Product	Category	Composition
1.	Inviro Alcohol based Surface and Equipment Disinfectant	Surface and equipment disinfectant	Composition Each 100 gms contains Ethanol (Denatured) 10 gms, 2-Propanol - 9 gms, 1-Propanol - 6 gms
2.	Inviro Pro Aldehyde free 4th generation QAC Surface Disinfectant Concentrate	Surface Disinfectant with perfume	Composition Each 100 gms contains Didecyl dimethyl ammonium chloride 7 gms, Perfume
3.	Inviro Aldehyde free 4th generation QAC Surface Disinfectant Concentrate	Surface cleaner/ disinfectant	Composition Each 100 gms contains Didecyl dimethyl ammonium chloride - 7 gms
4.	Myscan 2%	Instrument & Scope Disinfectant	Composition Glutaraldehyde Solution 2%
5.	Myscan 2.45%	Instrument & Scope Disinfectant	Composition Glutaraldehyde Solution 2.45% w/v
6.	Myscan OPA	Instrument & Scope Disinfectant	Composition Orthophthalaldehyde -0.55%, Colour- Alzarine Cynine Green F
7.	Mynox Linen Disinfectant	Hospital Fabric & Linen Disinfectant	Ingredient: Didecyl dimethyl ammonium Chloride, Citric acid, Ortho-Phosphoric acid, Enzyme, Fragrance, Purified water



29 Meril

Subsidiaries

12 Academies

Globally

10000+
EMPLOYEES

250+
PRODUCTS

158+
COUNTRIES



 **Manufactured by:**

Meril Medical Innovation Pvt. Ltd.

Ground & First Floor, Survey No. 173/4 &
First Floor, H1-H3, Meril Park,
Survey No. 135/2/B & 174/2, Muktanand Marg,
Chala, Vapi - 396 191, Gujarat, India

Customer Care Contact:

Email: askinfo@merillife.com

Marketed By:

Meril Life Sciences India Pvt. Ltd.

6th Floor, B wing, Bonanza Building,
Sahar Plaza, Chakala JB Nagar,
Nr Kohinoor Hotel, Nr Chakala Metro Station,
Andheri East. 400059